

How to change your cartridge type

Setting the Cartridge Type on your easypod 2 device

Changing the Cartridge is only required when a different concentration of cartridge is used, such as changing from a 6mg Saizen (Somatropin) cartridge to a 12mg or 20mg cartridge.

	Cartridge content	concentration	cartridge
B	Saizen® 6 mg	5.83 mg/ml	Solution for injection 
A	Saizen® 12 mg	8.00 mg/ml	Solution for injection 
A	Saizen® 20 mg	8.00 mg/ml	Solution for injection 

N.B. this should be performed when there is **No** cartridge in the device and after the new daily dose has been changed. Please see changing dose instructions in the download section, www.mysupport-resources.com

Open the clinical setting menu

To change a clinical setting:

1. Choose **Menu > Clinical settings** and press **Open** to select.
2. The warning message '**changes can affect treatment**' is displayed.
3. The easypod beeps three times to highlight the warning.
4. Press **OK** to continue (or **Exit** to return to the main menu). The Pin code screen is displayed

Your pin code is: _____

If you do not have your device pin please contact your clinical team
or

Merck Medical UK (Part of the MySupport service) Tel: 0208 818 7373

Merck Medical IRE (Part of the MySupport service) Tel: 01-4687590

5. Use the **↑** button to change the number.
6. Press **OK** to select the number you need.
7. Repeat steps 3 and 4 until you have selected your PIN code and press **OK**. The **Clinical settings** menu will be displayed.

How to change your cartridge type

Setting the Cartridge Type on your easypod 2 device

Changing the Cartridge Type

1. Choose **Injection Settings** and press **change**
2. Select **Cartridge** and press **Change**
3. Select the cartridge: A – 8.00 mg/ml
B – 5.83 mg/ml

See the previous table to confirm the correct type is being selected

4. Once selected press **OK** – your cartridge type is now changed

N.B. Should you receive a caution message here, please check you have the correct daily dose set in the device.